

# Poultice

Animal dressing that may be used either warm and wet, cool and wet or dry to suit your horses needs.

## DESCRIPTION

Animal dressing that may be used either warm and wet, cool and wet or dry.



### Usage:

Cut dressing to size and apply to affected area with plastic backing away from skin using one of the following methods:

- **Warm and Wet:** Allow boiled water to cool to 38°C, soak briefly, lightly squeeze and apply dressing.
- **Cool and Wet:** Chill boiled water, soak briefly, lightly squeeze and apply dressing.
- **Dry:** Apply dressing directly to affected area.

Dressings should be changed at least every 12 hours. Hold dressing in place using a Plusvital Wrap cohesive 20 cm x 40 cm bandage (avoid applying too tightly).

### Warning:

Consult a veterinary surgeon if Plusvital Poultice is to be used in conjunction with medical preparations.

### Directions for Use:

- See usage note above.



muscle support



water resistant



easy to apply

## KEY FEATURES

- Highly absorbant
- Muscle support
- Moisture barrier
- Easy to apply

### Available Sizes:

Sold in boxes of 10

20 cm x 40 cm